GERÄTEDOKUMENTATION

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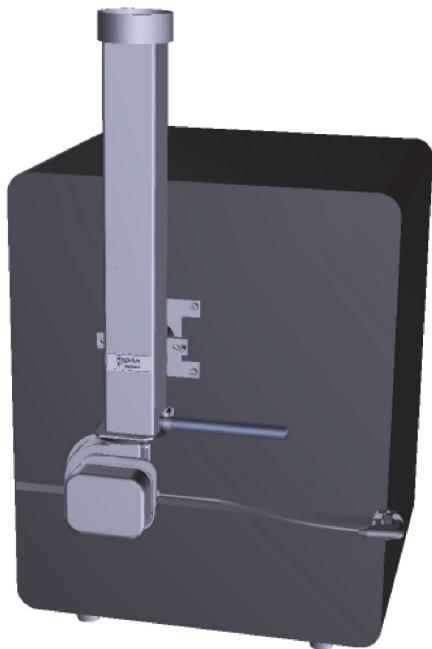
Regulus

DOC 86116 / 00

- de** Deutsch
- en** English
- fr** Français
- es** Español
- ru** Русский


Regulus
40300101

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
Regulus

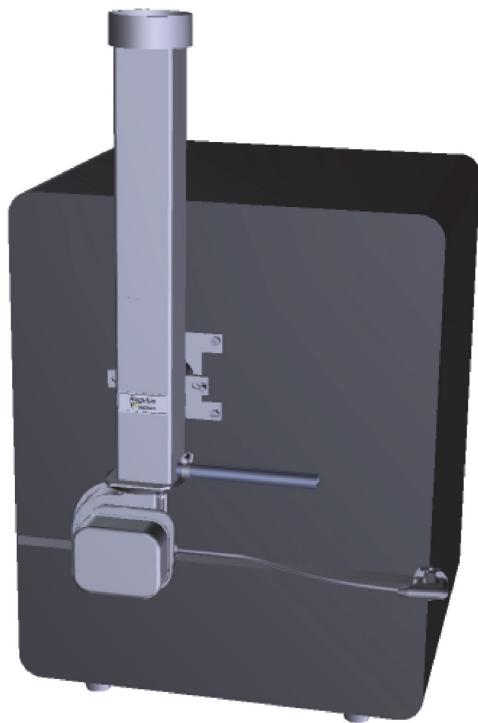de Deutsch

Regulus

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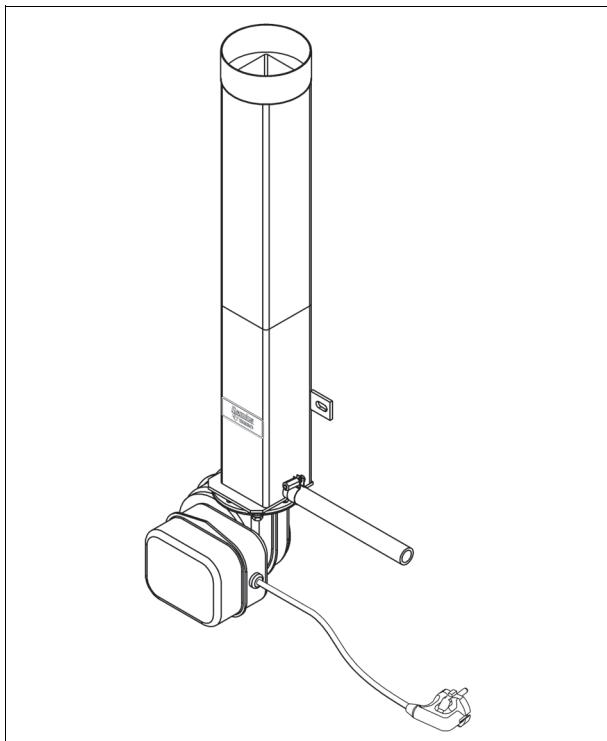
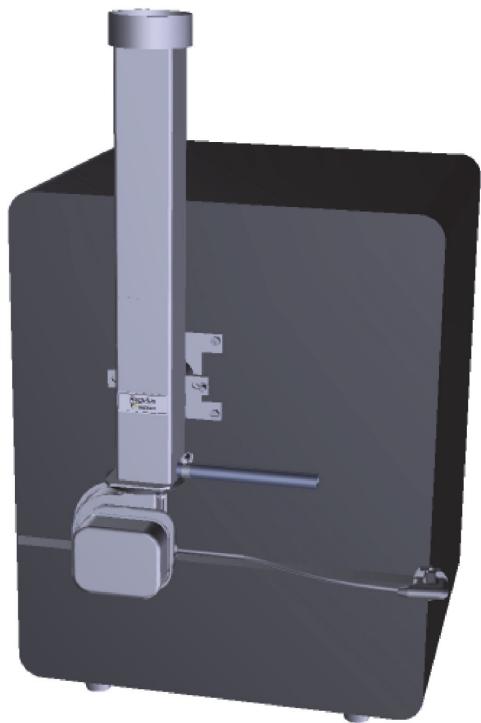
Translation of the original instructions

Regulus

en English

Regulus

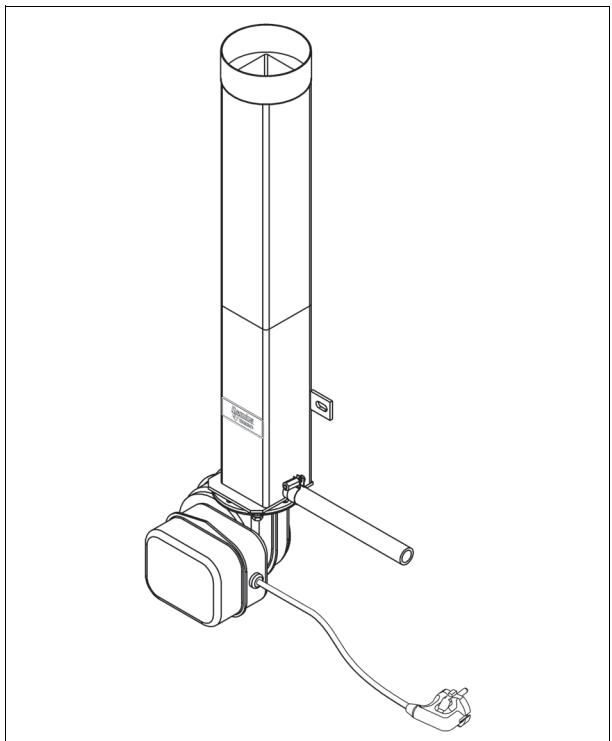This unit documentation is part of the unit and must be enclosed when selling or passing on the unit.

- The unit has been designed solely for use in dental laboratories and comparable establishments for research, commercial and training purposes. The unit must only be operated by dental professionals, or they must provide supervision if trainees or other persons operate unit.
- The operating instructions must be read and understood before the device is used. This particularly applies to the **Safety information**. The warranty is void in cases of damage which is caused by non-observance of the operating instructions. We will also not accept liability for any resulting consequential damages.
- Symbols used

This symbol indicates very important information. Failure to comply with it may result in personal injury.

Hazard symbol Environmental hazard

This symbol indicates that safety gloves have to be worn.

Table of contents

General information.....	13
Safety instructions.....	15
Unit description	17
Scope of delivery.....	17
Technical data	17
Installation	18
Care and maintenance.....	19
Signs, labels and stickers.....	19
Disposal	20
Certificate of Conformity.....	Appendix

General Information

Importance of the operating Instructions

These operating instructions cover all the information that is necessary according to the relevant regulations for the safe operation of the device described herein.

The operating instructions are a part of the machine. The operating instructions should therefore

- always be kept at hand near the device until the device is disposed of,
- and should be passed on with the device when it is purchased, sold, or lent.

Contact the manufacturer if you do not understand something in the operating instructions.

We welcome any suggestions or criticism. Please feel free to notify us. Your effort will help us make the operating instructions more user-friendly and respond more effectively to your wishes and needs.

Target group

This document is directed toward everyone who works with this device or performs service tasks that are described in this document.

Contact information

BEGO Bremer Goldschlägerei Wilh. Herbst GmbH & Co. KG
Wilhelm-Herbst-Str. 1
28359 Bremen • Germany
Telephone: +49 421 2028-0
Service-Hotline Tel.: +49 421 2028-270
www.bego.com

© Copyright by BEGO Bremer Goldschlägerei Wilh. Herbst GmbH & Co. KG

This user guide is protected by copyright. All rights reserved.

No part of this document may be reproduced without the express written permission of BEGO Bremer Goldschlägerei Wilh. Herbst GmbH & Co. KG!

Conventions

These operating instructions contain references to residual hazards, important user tips and handling instructions that are identified with the following symbols and words.

⚠ DANGER

This reference identifies hazards that can cause serious bodily injury or death if the relevant hazard instructions are not followed or not followed properly.

⚠ WARNING

This reference identifies hazards that can cause bodily injury or property damage if the relevant hazard instructions are not followed or not followed properly.

⚠ ATTENTION

This reference identifies only those hazards that are potential damaging to property and the environment.

NOTE

This symbol identifies user tips and particularly useful information. It helps you optimally exploit all the functions of your machine.

1. Sequential actions are described in sequentially numbered paragraphs.
- Cross-references are identified with this symbol.

Warranty and Liability

Our general terms and conditions of sale and delivery apply. These are made available to the customer on completion of the contract, at the latest. Warranty and liability claims in the event of physical injury or damage to property are invalid if they are caused by one or more of the following:

- Improper use of the device
- Improper installation, putting into operation, operating and maintenance of the device;
- Operating the device with safety and protective facilities that are defective, improperly installed or inoperative
- Failure to heed the information in this manual concerning the transport, storage, installation, putting into operation, operation and service and maintenance of this device;

- Unauthorized structural modifications to this device;
- Inadequate monitoring/checking of parts of the device that are subject to wear;
- Improperly performed repairs;
- Catastrophes caused by alien elements or force majeure.

Safety instructions

Regulus is exclusively intended for the extraction of kiln gases from BEGO preheating ovens. Any other use is considered to be improper. We shall not be liable for any damage resulting from improper use.

The safety instructions must be followed in order to prevent personal injury and damage to the machine. We shall not be liable for damage due to failure to comply with the safety instructions.

General Safety Instructions

Place of use - The unit has been designed solely for use in dental laboratories and comparable establishments for research, commercial and training purposes.

The unit must only be used in dry rooms.

Operator and operating personnel - The unit must only be operated by dental professionals, or they must provide supervision if trainees or other persons operate unit.

Wear the protective clothing prescribed for use in dental laboratories and observe the rules of conduct.

The unit may only be operated by staff members who are familiar with and follow the instructions of this operating manual.

The operating manual must be easily accessible when the machine is operated. Observe the national accident prevention regulations in addition to this operating manual.

Handling the machine - Prior to the electrical connection check whether the specifications on the rating plate correspond to the power supply network. In the case of uncertainty, consult an electrician.

The unit is categorised under protection class 1 and may only be connected to Miditherm 100 MP/ 200 MP.

Modifications of the machine are not permitted.

Always keep signs and stickers such that they are easy to read. They must not be removed.

Check the machine and leads for damage regularly. The machine must not be operated if it has any defects that may result in risks for the employees or third persons.

Protect the leads against heat, oil and sharp edges. The leads must not be used to carry the machine or to pull out the mains plug.

Repairs - Repairs may be carried out solely by BEGO customer service or by persons authorised by BEGO customer service.

Metallic housings must be grounded properly so as to prevent them from carrying current. Otherwise they are extremely dangerous because the housings may be live due to damage to the machine! Since a check to ensure a de-energized state is mandatory after every intervention in the machine (Germany: check in accordance with DIN VDE 0701-1), only qualified electricians may open the machine!

Only BEGO replacement and wearing parts may be used.

Care and maintenance - The unit has been designed for a service life of 10 years from the date of manufacture. No liability is accepted for damages arising from operation of the unit after this period. Switch off the machine and pull out the mains plug before performing any care, cleaning or maintenance work.

Only use dry or slightly moist cloths for cleaning. Do not spray the machine with water or immerse it in water.

Special safety instructions regarding this machine

⚠ WARNING

Installation

- Only connect Regulus to cold pre-heating ovens.
- When laying cables and hoses, ensure that they are protected from hot components!
- The machine must only be connected to the socket provided on the rear of BEGO pre-heating ovens.
- Drain condensation water into a suitable container using the supplied drain hose and empty the container regularly.

⚠ WARNING

Burn hazard from hot parts!

- The housing and emerging vapours may be hot.
- Allow the machine to cool down before carrying out any cleaning or maintenance.

⚠ WARNING

Warning of harmful or irritant substances

Manufacturers of other investment materials or waxes do not disclose additional ingredients for reasons of confidentiality. It is therefore not possible to make general statements about the residues that are released and their respective composition.

⚠ WARNING

Danger from waste gases

- The vapours extracted from the oven may be harmful to health. Place the preheating oven with Regulus under an extraction hood, if possible.

Always discharge vapours outdoors.

Unit description

Regulus

The Regulus extracts vapours from the furnace compartment rapidly and thoroughly using the injector principle.

Scope of delivery

Regulus

200-240 V, 50-60 Hz REF 25750

with:

- Fan with rubber flange and 2 rubber sockets (assembled)
- Oven connector (2 parts)
- 1 Condensation drain hose with clip
- Mounting screws
- Operating instructions

NOTE

The spare parts list is appended.

When ordering spare parts please quote the REF, quantity and equipment serial no (SN).

Technical data

Height.....	600 mm	Power consumption.....	23 VA
Width.....	160 mm	Noise emission	< 70 dB (A)
Depth	140 mm	Air volume	max. 27 m ³ /h
Weight.....	3.4 kg	Pressure differential	max. 70 Pa
Nominal voltage	200-240 VAC, 50-60 Hz		

Installation

⚠ WARNING

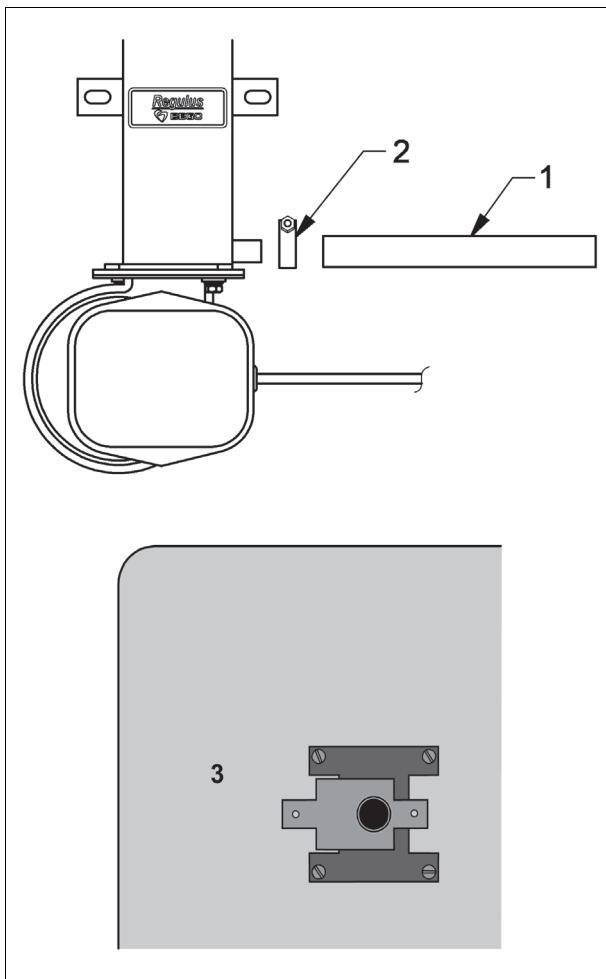
- Only connect Regulus to cold pre-heating ovens.
- When laying cables and hoses, ensure that they are protected from hot components!
- The machine must only be connected to the socket provided on the rear of BEGO pre-heating ovens.

⚠ WARNING

- Drain condensation water that is collecting in the supplied extraction hose into a preferably heat-resistant, sealable container and dispose of it regularly.
 - Place the preheating oven with Regulus under an extraction hood, if possible.
- Always discharge vapours outdoors.**

1. Push on the hose (1) and secure with the hose clip (2).
2. Push the two parts of the connector together to make it as long as possible so that the exhaust fumes/gases are directly channelled into the exhaust hood.
3. Screw the assembled fan onto the connector on the rear panel of the oven (3).
4. Insert the plug into the socket at the rear of the oven.

NOTE

The Regulus extraction pipe can be extended by up to 2 m with normal commercial 80 mm dia. pipes.

Always discharge vapours outdoors!

Care and maintenance

⚠ DANGER

Warning of electric shock! Danger to life!

Switch off the machine and pull out the mains plug before performing any care or maintenance work.

In the case of power connection without mains plug: remove fuses, secure machine so it cannot be switched on again and check that it is de-energized.

⚠ WARNING

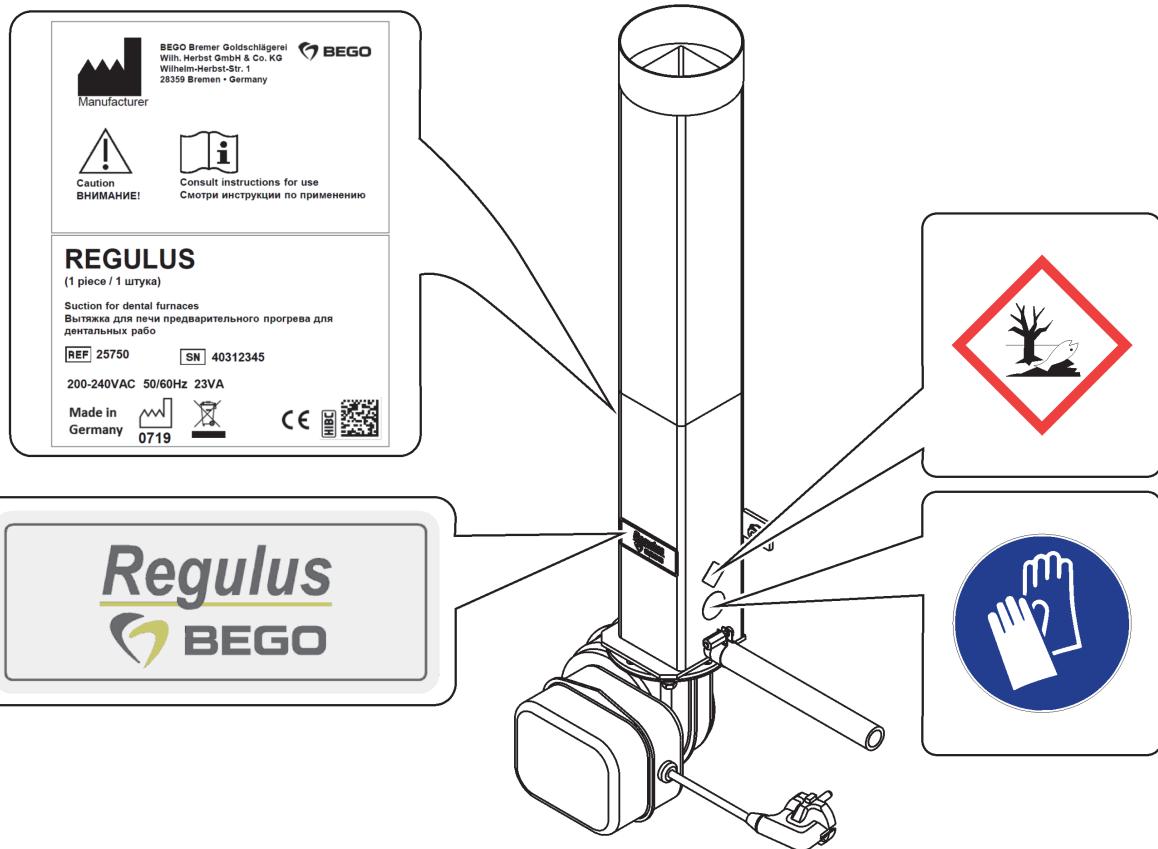
Burn hazard from hot parts!

Carry out maintenance and cleaning work before working or only after the machine has cooled down.

1. The extraction pipe must be cleaned roughly every 14 days for furnaces that are used 2 or 3 times a day.
2. In all other cases, check and clean the extraction pipe once a month.

Signs, labels and stickers

Disposal

Instructions for disposing of the device

(Applicable only within the European Union)

The adjacent symbol on the ID plate of the BEGO device indicates that the device, in accordance with the European directive on waste electrical and electronic equipment, may not be disposed of as normal domestic waste.

As a customer, you contribute to the protection of the environment when you dispose of the device correctly.

Disposal in Germany

BEGO offers you a disposal solution for all BEGO devices sold and put into operation in Germany subsequent to August 13, 2005. Please contact us when it is time to dispose of the device.

Disposal in other countries of the European Union

Please contact the company from which you purchased the BEGO device when it is time to dispose of it. They will provide you with information concerning correct disposal in your region.

Disposal of the condensate

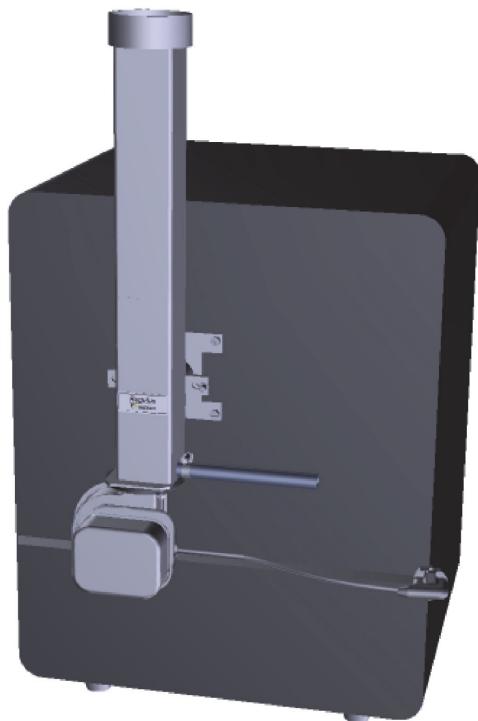
Pollution of the environment and the ground water from improper disposal.

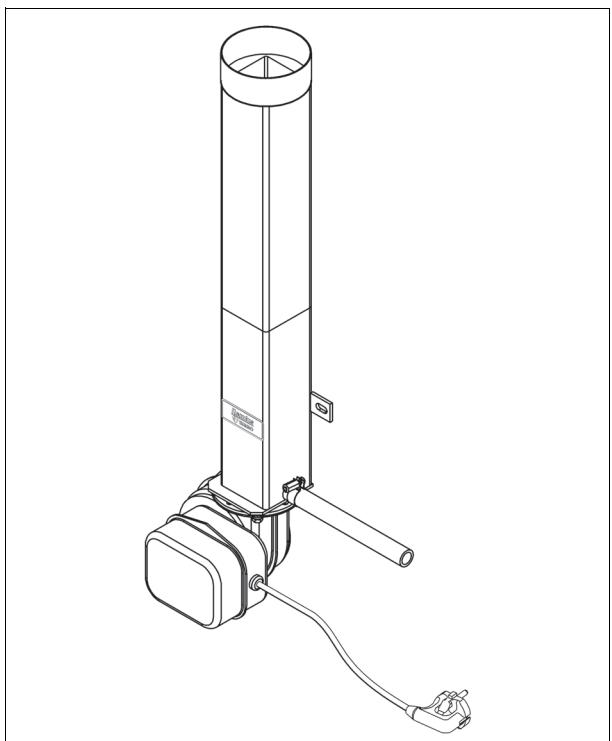
Observe the local regulations and guidelines for the disposal of the condensate that are applicable in the country where the device is operated.

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