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DE Muffelformer

Muffelformer

- 4 kleine Muffelformer (Farbe rot) inklusive Hartgummiplatten
- 4 große Muffelformer (Farbe blau) inklusive Hartgummiplatten

REF 52390

REF 52400

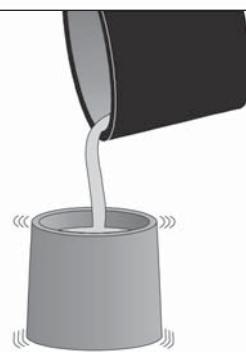
1

2

3

4

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EN Mould former

Mould former

- 4 small mould formers (red) including hard rubber plate
- 4 large mould formers (blue) including hard rubber plate

REF 52390

REF 52400

1

- ① Place model on hard rubber plate and fix it with wax.
- ② Cover with the mould former.
- ③ Fill with investment material on the vibrator.
- ④ After setting remove hard rubber plate and press out mould.

The life of the BEGO mould former can be considerably increased

- by lubricating it for instance with vaseline,
- by removing the mould immediately after setting,
- by cleaning it at once with water.

2

3

4

* For easy processing, the BEGO combi duplicating flask (order no. 52090) can be used. Due to the special form of this flask, a model with fitting base plate will be produced during duplicating with gel. Thus, the hard rubber plate of the mould former is no longer necessary.

Warranty

Whether given verbally, in writing or by practical instructions, our recommendations for use are based upon our own experience and trials and can only be considered as standard values. Our products are subject to a constant further development. Therefore alterations in construction and composition are reserved.

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